**HOSPITAL MELAKA**

## **PROSEDUR KUALITI**

**Pengurusan Spesimen Bukan “Urgent” Dan Keputusan  
Ujian Pesakit**

<b>No. Dokumen</b>	<b>PK - HM - SK- 01</b>	<b>No. Keluaran : 01</b>
--------------------	-------------------------	--------------------------

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## 3. RUJUKAN

- 3.1 Laboratory Handbook 2017, Jabatan Patologi, Hospital Melaka. (<http://hmelaka.moh.gov.my>)
- 3.2 Manual Kualiti Jabatan Patologi Hospital Melaka (HM/JP/QM-01)
- 3.3 Prosedur Kerja / Arahan Kerja Unit – Unit, Jabatan Patologi Hospital Melaka
- 3.4 Akta Dokumen Rekod Perubatan 2007
- 3.5 College of Pathologists Version 1/2005 – “Guidelines on Retention of Pathology Records and Materials” Medical Laboratories – Particular Requirements for Quality and Competence. (MS ISO 15189 – 2014: Clause 4.3)

 <b>HOSPITAL MELAKA</b>	<b>PROSEDUR KUALITI</b> <b>Pengurusan Spesimen Bukan “Urgent”  dan Keputusan Ujian Pesakit</b>	<b>No. Pindaan: 00</b> <b>Tarikh: 15.01.2018</b>
--	---	---

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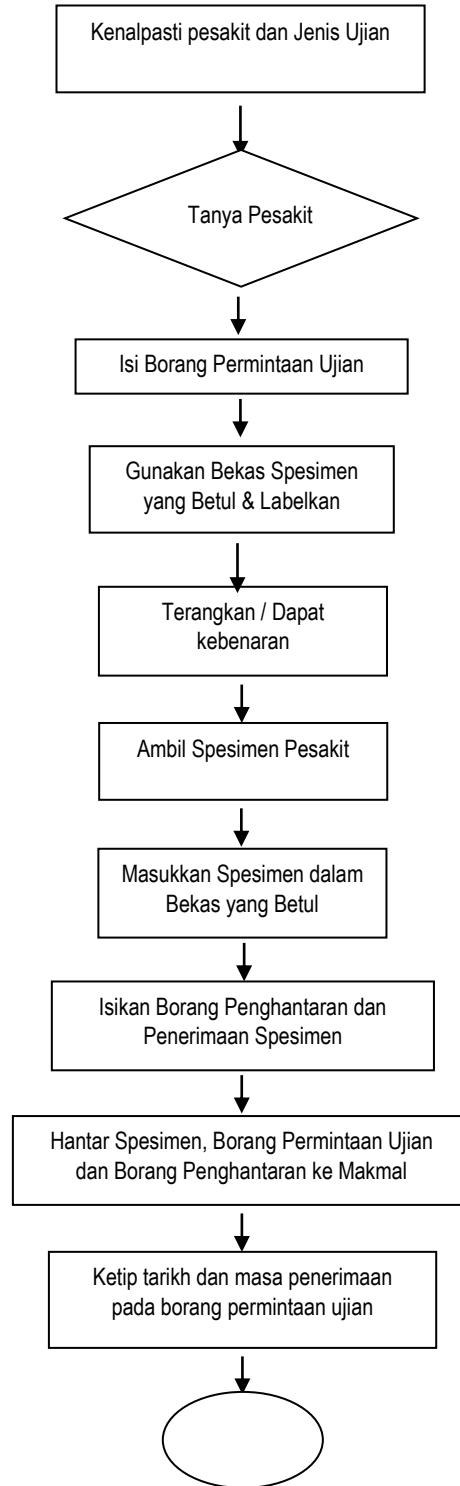
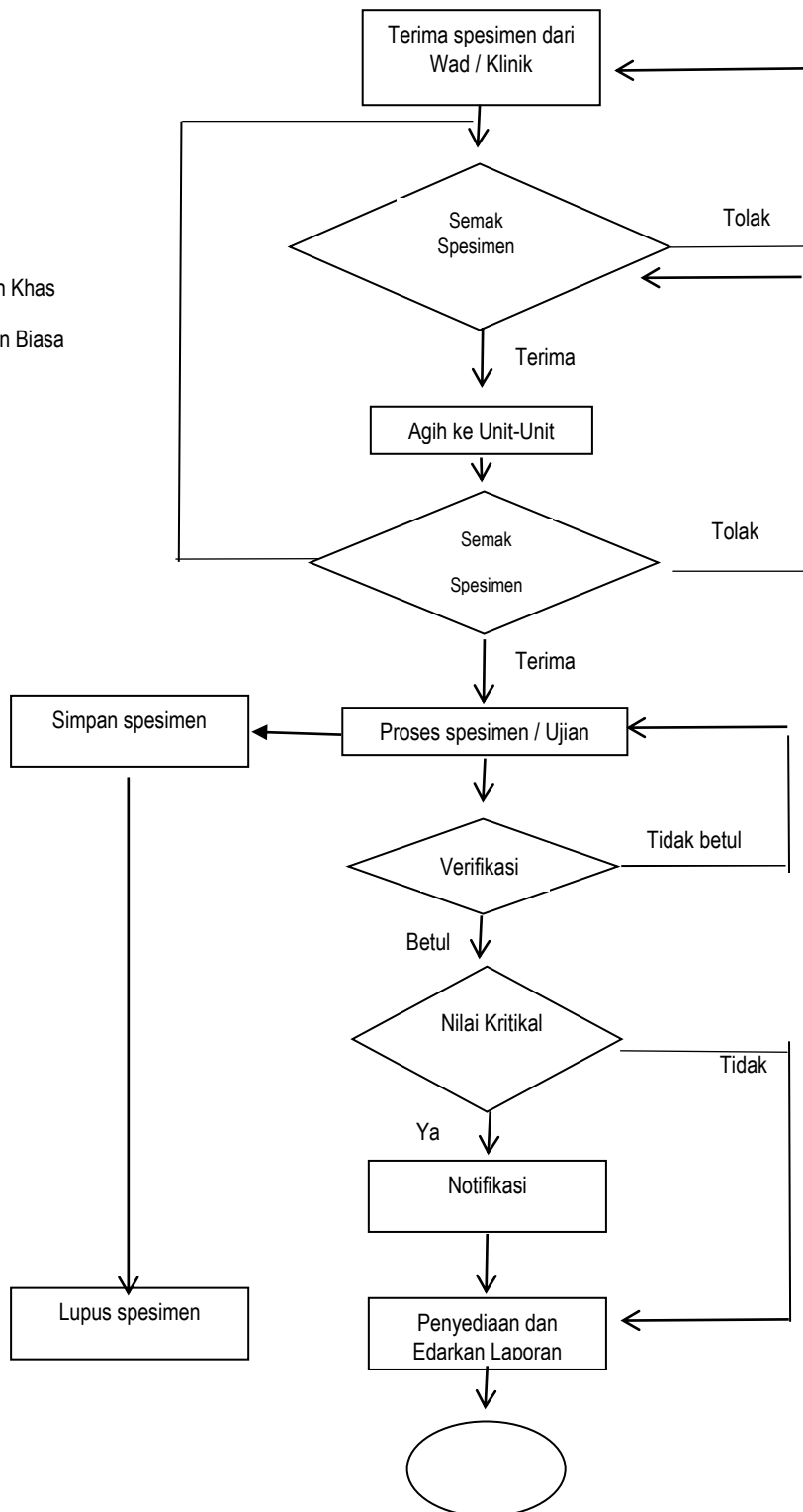
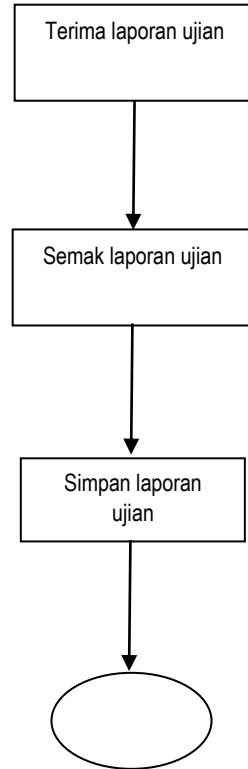
\* Mengikut Akta Dokumen Rekod Perubatan 2007 / College of Pathologists Version 1/2005 – “Guidelines on Retention of Pathology Records and Materials” / Prosedur Kerja Unit Masing-masing.

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MSISO/HM/JP/QP-004/01

Keluaran: 02

**Guideline For Main Counter Specimen Reception / Rejection**

1. Received specimen and request form.
2. Sign and date “Borang Penghantaran / Penerimaan Spesimen”
3. The following specimen are send directly to the respective unit by the ward attendant / authorized personnel.
  - a. Blood Gas Specimen, Forensic specimen, Lactate and Ammonium
  - b. Urine For Abused Drugs
  - c. Medico-legal specimen
  - d. Specimen ‘Glucose-6-phosphatase dehydrogenase’
  - e. PAP Smear from District Hospital and health clinic.
  - f. Renal biopsy & skin for Direct Immuno-Fluorescence
  - g. CSF specimen
  - h. Dengue Combo Specimen
  - i. MERSCoV Specimen
  - j. TB Gene Xpert
  - k. Hb Analysis Saringan Thalassemia Pelajar Tingkatan 4
4. Check specimen and request form according to the following acceptance criteria:
  - 4.1 a) Name, Identification Card Number, Test requested are tally.  
b) Barcode label on specimen and request form are tally.
  - 4.2 Appropriate request forms used with adequate information.
  - 4.3 Information on the request form and specimen are tally.
  - 4.4 The request form bears the doctor’s signature.
  - 4.5 Correct specimen container are used.
  - 4.6 Specimen and request form are segregated. “Biohazard” specimens are sent in “Biohazard” bags and “Routine” specimens are sent in normal plastic bags.
5. If the request fulfills the acceptance criteria, the specimen and request forms are sorted and sent to the respective unit.
6. “STAT” / URGENT requests are send to the respective units immediately and the laboratory staffs in the respective unit are notified.
7. If the specimen or / and request form do not meet the acceptance criteria, fill up the “Laboratory Rejection Form” in duplicate.
8. Inform the ward of the rejection through phone, especially for STAT cases, record the name of the receiver & time of informing onto the rejection form.
9. Dispatch the Rejection Form and the laboratory request form via:
  - » Ward attendant who sent the specimen
  - » Pneumatic tube
  - » Pigeon Hole
10. File a copy of the Rejection Form.

 <b>HOSPITAL MELAKA</b>	<b>PROSEDUR KUALITI</b> <b>Pengurusan Spesimen Bukan “Urgent”  dan Keputusan Ujian Pesakit</b>	<b>No. Pindaan: 00</b> <b>Tarikh: 15.01.2018</b>
--	---	---

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(All relevant staff must note the date and time of each process and initial the entry)

Name: .....R/N:.....Ward:.....

Process	Date	Time (hh:mins)	Whom to complete and initial
1. Collection of specimen			Doctor/nurse
2. Arrival in laboratory			MLT/Receptionist
3. Preliminary microscopic examination report sent out			MLT
4. Preliminary microscopic examination report reach the ward. #			MLT

Culture result :Positive / Negative

Organism isolated :.....

Antigen detection result :Positive / Negative / Not Done

Antigen detected :.....

If time taken from collection to reporting exceeds 3 hrs, give reason for delay:

.....  
.....  
.....

\*Delete accordingly

# Same as No. 3 if report send by telephone or personal communication.

Notes :

1. Time motion form (CSF) should accompany all CSF specimens received.
2. Preliminary report could be by telephone, written or personal communication.

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1. Borang Permintaan Perkhidmatan Patologi (Pathological Service Request Form) - PER-PAT 301
2. Laboratory Request Form for Dengue and Flavivirus- MKAK-BPU-D02(rev\_Nov 2015)
3. Laboratory Request Form for Novel Coronavirus Investigations
4. Cytology PAP SMEAR- PS 1/98-Pindaan 2007
5. Therapeutic Drug Monitoring (TDM) Request Form- MS ISO HM/ FAR 14 (“YELLOW” form)